FlurAbsorb Pro-L / FlurAbsorb Pro-S

Anaesthetic Gas Filter



FlurAbsorb Pro-L 1026096

FlurAbsorb Pro-S 1026094

	Set-up Quick Start	2
EN	Instructions for Use	4
DE	Gebrauchsanweisung	5
FR	Mode d'emploi	6
ES	Instrucciones de uso	7
IT	Istruzioni per l'uso	8
SV	Bruksanvisning	9
NO	Brugsanvisning	10
DA	Brugsanvisning	11
FI	Käyttöohjeet	12
EL	Οδηγίες χρήσης	13
NL	Gebruiksaanwijzing	14
PL	Instrukcja użytkowania	15
PT	Instruções de Utilização	16
RU	Руководство по эксплуатации	17
EE	Kasutusjuhend	18
SL	Navodila za uporabo	19
CZ	Návod k použití	20
SK	Pokyny na používanie	21
TR	Kullanma talimatları	22
BG	Инструкции за употреба	23
HR	Upute za uporabu	24
HU	Használati útmutató	25
SR	Uputstvo za upotrebu	26
RO	Instrucțiuni de utilizare	27
AR	مادخست الة تاميل عت	28
KO	사용 설명서	29
JP	取扱説明書	30
ZH	使用说明	31

FlurAbsorb Pro-L / Pro-S Set-up Quick Start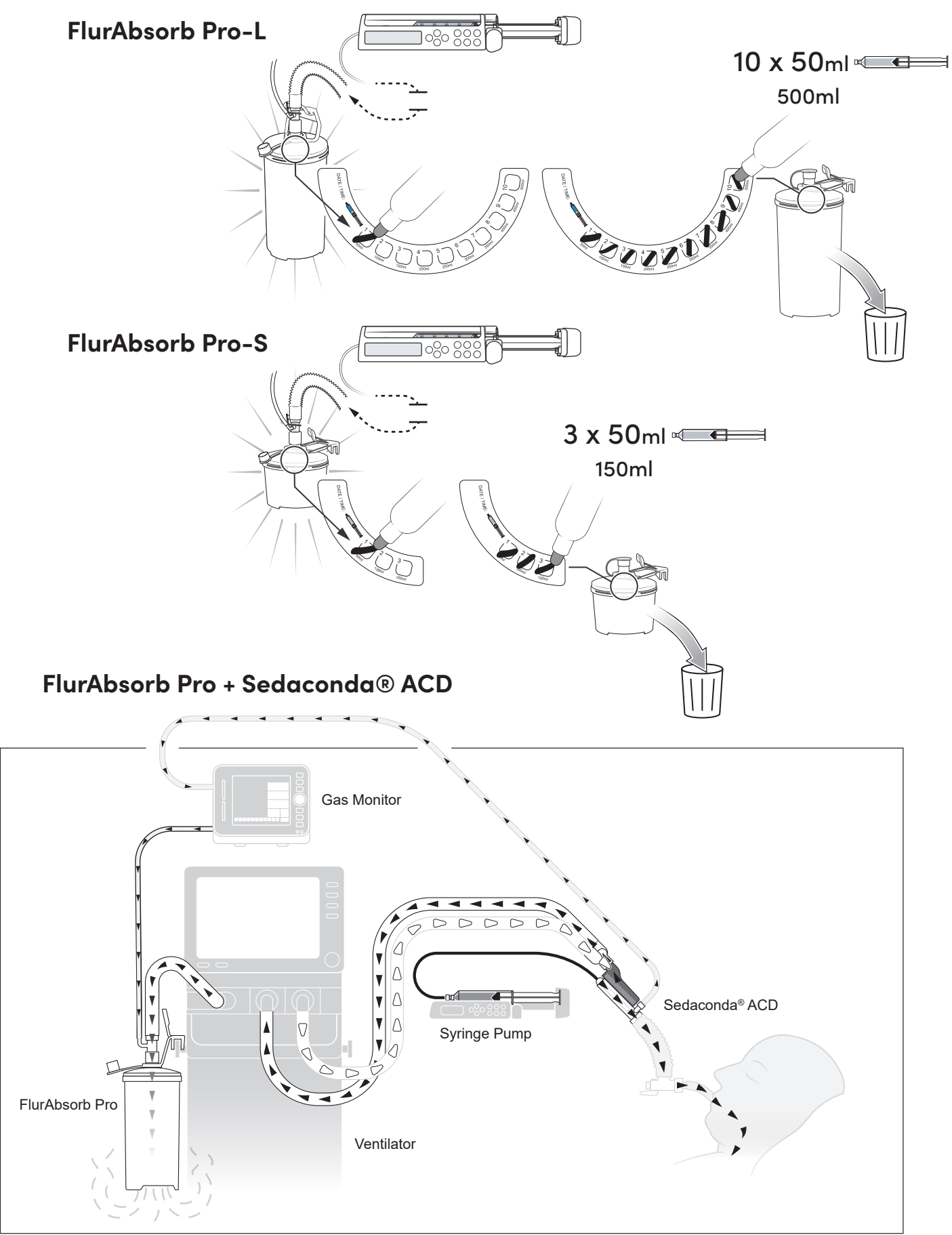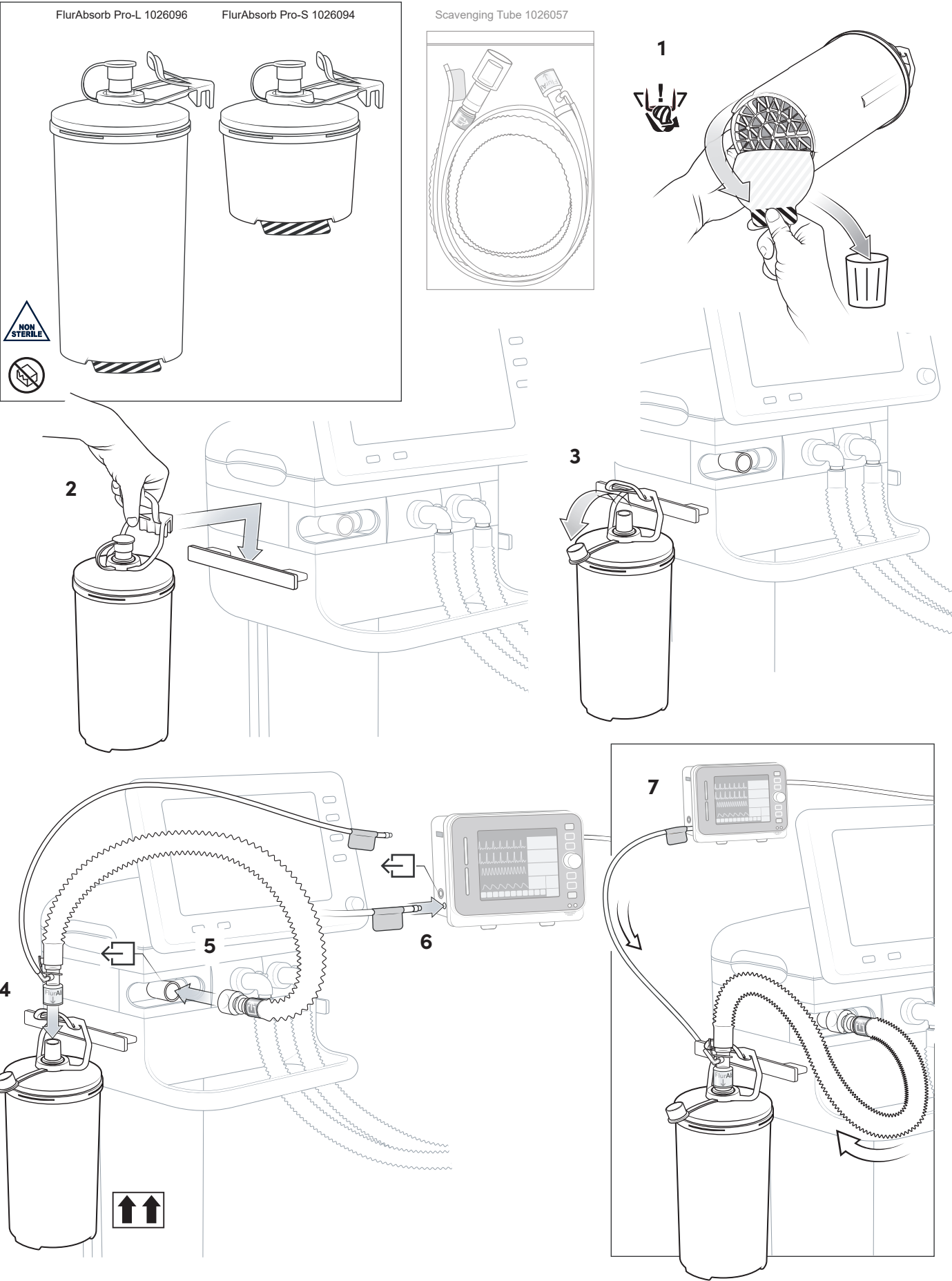

1. INTENDED USE

The FlurAbsorb Pro is a passive gas scavenging device intended for the absorption of volatile anaesthetic gases from a ventilator exhaust. The purpose of FlurAbsorb Pro is to prevent ambient pollution and limit user exposure to volatile anaesthetic gases. The FlurAbsorb Pro-L (1026096) and FlurAbsorb Pro-S (1026094) are designed with absorption capacities of 150 mL and 500 mL of volatile anaesthetic agents, respectively. The FlurAbsorb Pro is primarily intended for use within the Sedaconda® system. The FlurAbsorb Pro is intended for use by trained healthcare professionals in an appropriate healthcare setting.

2. SETUP

1. Remove the FlurAbsorb bottom seal from the base of the container.
2. Using the integrated hook & handle, mount the FlurAbsorb Pro on a medical accessory rail.
 - If no medical accessory rail is available, the FlurAbsorb Pro can be placed upright on the floor, next to the ventilator.
3. Remove the 22mm Cap from the FlurAbsorb Pro.
4. Connect a gas scavenging tube 1026057 between the ventilator exhaust and the filter's 22mm connector.
5. Track the number of syringes and volume of anaesthetic agent by marking the guided label on top the FlurAbsorb Pro.
6. When the maximum absorption capacity is achieved, replace the FlurAbsorb Pro with a new device:
 - Disconnect the gas scavenging tube.
 - Close the 22mm Cap.
 - Dispose of the FlurAbsorb Pro according to hospital protocols.

3. CLEANING DEVICE

To clean FlurAbsorb Pro, carefully wipe with alcohol. Avoid removing any markings made on the device to track the volume of anaesthetic absorbed.

4. GENERAL WARNINGS AND INFORMATION

- Remove FlurAbsorb seal (bottom sticker) before use.
- Ensure the FlurAbsorb Pro is always used in an upright position.
- Do not use FlurAbsorb Pro beyond its stated storage capacity:
 - FlurAbsorb Pro-L: 500 mL of volatile anaesthetic agent.
 - FlurAbsorb Pro-S: 150 mL of volatile anaesthetic agent.
- Use caution when using FlurAbsorb Pro with active humidification. Absorption capacity may be affected by excessive moisture.
- When using FlurAbsorb Pro with active humidification, it is recommended to insert a dehumidifier on the ventilator expiratory limb.
- Replace the FlurAbsorb Pro immediately in case of occlusion.
- Disconnect the FlurAbsorb Pro in case of ventilator alarm due to exhaust port occlusion.
- Do not use the FlurAbsorb Pro if the device is damaged or broken.

In case of a serious incident please contact the competent authority and the legal manufacturer, Sedana Medical Ltd.

Sedana Medical Ltd.
Unit 2A, The Village Centre
Two Mile House, Naas
Co. Kildare, W91 PWH5
Ireland

✉ info@sedanamedical.com

5. SYMBOLS

The following symbols are used in the Instruction for Use (IFU) or on the product itself.

Symbol	Description
	Remove bottom seal before use
	Keep upright
	Non-sterile
	Do not use if package is damaged
	Consult instructions for use
	In case of a serious incident.
	Keep in dry conditions
	Contents
	Date of manufacture
	Expiry date
	Legal manufacturer
	Made in Japan
	Medical Device (MDR Compliant)
	CE Mark

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Sedana Medical Ltd.
Unit 2A, The Village Centre
Two Mile House, Naas
Co. Kildare, W91 PWH5
Irland

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