

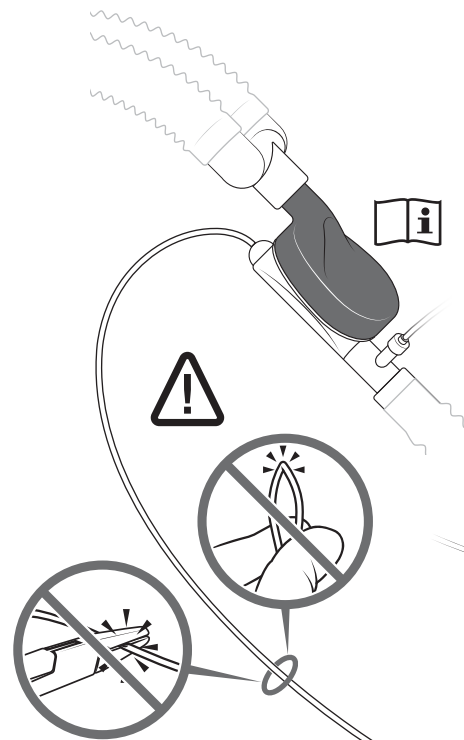
Sedaconda Syringe

REF 1026022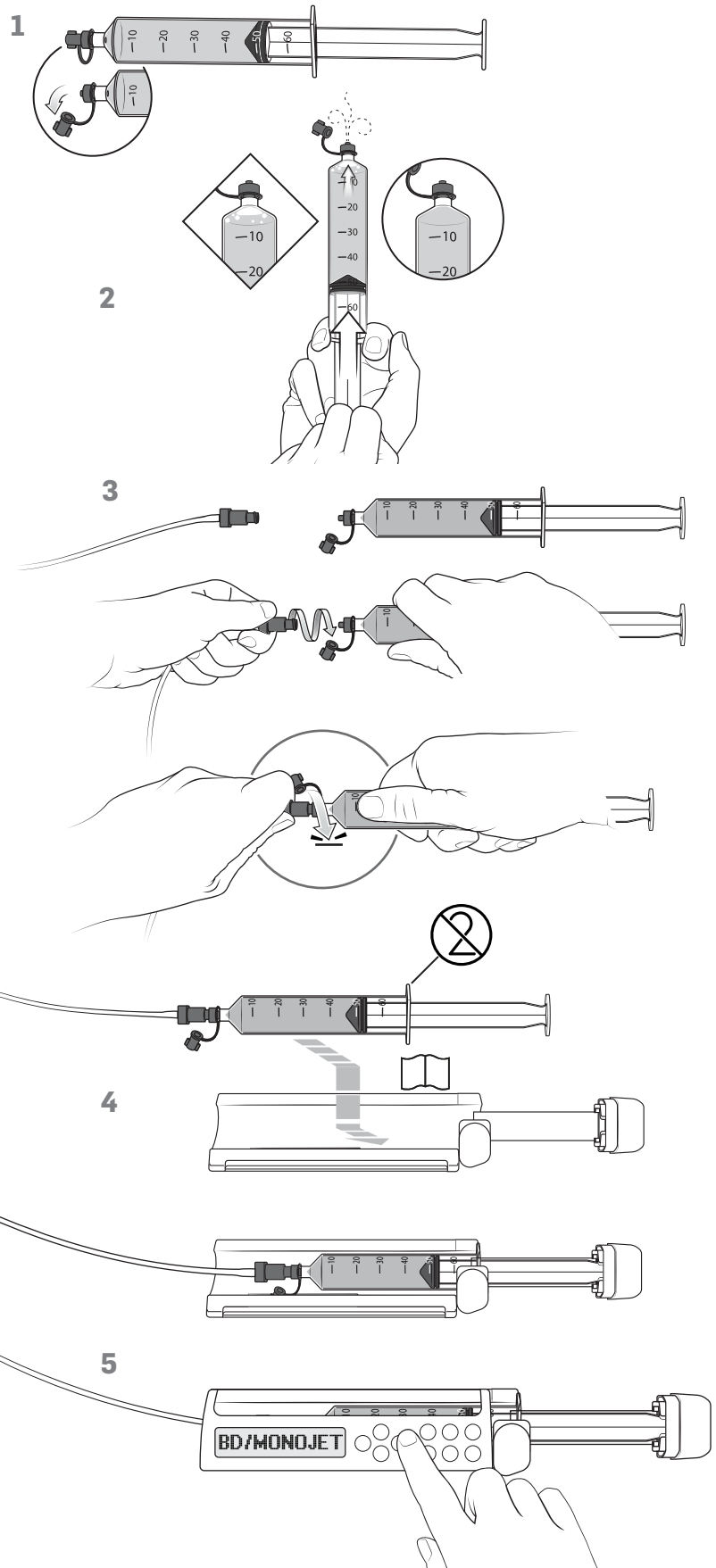

Sedana Medical Ltd.
Unit 2A, The Village Centre,
Two Mile House, Naas,
Co. Kildare, Ireland
W91 PWH5
www.sedanamedical.com

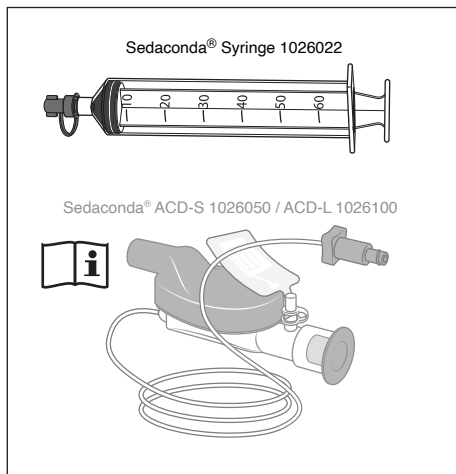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WARNING - differences in filled syringe storage environment temperature and use environment temperature can increase the risk of underdosage and overdosage.

Ensure all air is removed from the syringe before connection to the ACD. The presence of air in the syringe can lead to underdosage and overdosage.

Ensure syringe is tightened fully. Insufficient tightening can lead to leakage, underdosing and exposure to volatile anaesthetic gas.

Do not reuse! Reuse of syringe can increase risk of cross contamination within the use environment and lead to degradation in syringe performance and integrity, resulting in delivery inaccuracy and leakage of volatile anaesthetic.

Do not use the syringe if the integrity of the package is breached, visibly damaged or if there are foreign particles or substances present in the device .

Filling of the syringe shall be carried out in accordance with the respective filling adapter instructions for use.

Symbol	Description
	Not for IV use

Sedaconda® Syringe

Kasutusjuhend Návod na použitie Navodila za uporabo Návod k použití Kullanma talimatları Upute za uporabu Használati útmutató
Instrukcije za upotrebu Instrucțiuni de utilizare مداخلت سالا تامي لغت 사용 설명서 取扱説明書 使用说明 Инструкции за употреба

Sedana Medical Ltd., Unit 2A The Village Centre,
Two Mile House, Naas, Co. Kildare, W91 PWH5, Ireland

Date of issue: 19 Aug 2026
3000 309-2608/Rev.3

For the latest version of the IFU, please visit www.sedanamedical.com or scan the QR-code